



THUASNE

Achilles Walker



This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days.


This device is generally not intended for patients in excess of 250 lbs (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of re-injury.

Indications: Indicated for immediate application following injury to or reconstruction of the Achilles tendon.


Contraindications: Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.




1) Remove and upper calf cuff pad

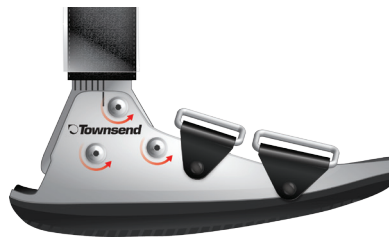


2) Loosen Allen screws on either side of walker shell to allow for ankle adjustment

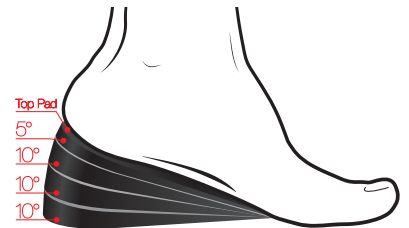


3) Select the appropriate wedge configuration (three 10 degree wedges and one 5 Degree Wedge)

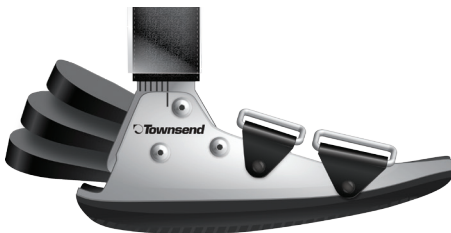

4) Place the wedges inside onto of inner sole

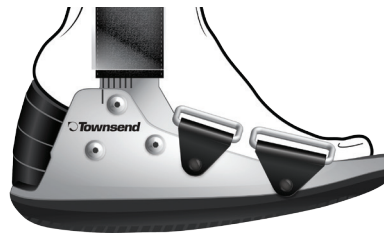

5) Place foot on wedges and adjust uprights mid-line to patient's calf

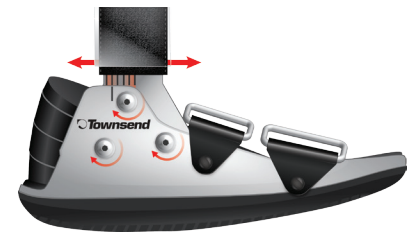

6) Once leg and ankle alignment has been achieved, tighten each Allen screw and remove upright plastic



7) Secure uprights to upper calf cuff pad



8) Apply and secure upright straps in the sequence and position suggested in illustration



9) Secure forefoot straps in the sequence suggested in the illustration.



10) When transitioning out of the walking boot. The wedges can also be used inside of the shoe for rehabilitation purposes. Select the desired wedge angle and insert into the shoe with the top cover wedge facing up. Remove the hook velcro tabs located on the top of the foam cover.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. If any additional pain or symptoms occur while using this device, address your healthcare professional.

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. For additional information, you can call Townsend Design Customer Service at 800.432.3466, visit us online or call your local sales representative.



EC

REP