Ligaflex® Rhizo

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Static thumb splint

Daumenorthese

Statische duimorthese

Ortesi statica del pollice

Ortesis estática de pulgar

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Statisk tommelfingerskinne

Staattinen peukalotuki

Statisk tumortos

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STATIC THUMB SPLINT

Properties:

- Immobilisation splint holding the thumb in an analgesic position.
- Product that fits the contours of the thumb perfectly due to wire stays with a spring effect.
- Inner lining in Thermytex fabric: breathable and antibacterial.
- Two Velcro straps for easy fitting of the product.

Indications:

- Immobilisation of the trapeziometacarpal joint in early osteoarthritis of the thumb.
- Pain at the base of the thumb.
- Peri-scaphoid joint disease.
- Following surgery
 - Thumb prosthesis
 - Trapeziectomy
 - Surgery for fractures at the base of the 1st metacarpal bone: Rolando or Bennett's fracture.
- Mild thumb sprain.

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

- 1 - Open the wrist straps.
- 2 - Slip the thumb into the splint.
- 3 - Adjust the wrist strap, pulling it through the tightening loop.
- 4 - Adjust the thumb strap.

In the event of discomfort, the stays can be shaped to fit the thumb web space.

Precautions:

To avoid any constriction problems, be careful not to over-tighten the Velcro straps.

Follow the advice of the health professional who prescribed or supplied the product. Consult your health professional if discomfort occurs.

Store at room temperature, preferably in its original packaging.

For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

See product label.

Do not use detergents, fabric softeners or aggressive products (products containing chlorine)

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Thuasne UK Ltd - Tunbridge Wells