LIGAFLEX[®] MANU

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Composition :

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WRIST AND THUMB IMMOBILIZATION SPLINT

Properties :

- Effective immobilisation of thumb and wrist (two adjustable X' ray transparent stays). (1).
- Improved comfort due to a 3D knit.
- Dual strap system ensuring even compression. (2)

Indications :

- Mild sprains of the wrist and/or thumb (functional treatment).
- Severe thumb sprains and scaphoid bone fractures (post-surgery treatment).
- Tendinitis and tendon disorders of the wrist and thumb.
- Inflammatory phases of rheumatic diseases.

Contraindication :

Do not apply the product in direct contact with broken skin.

Fitting instructions :

- Open the straps.
- Put the hand into the orthosis.
- Close the straps.
- Be careful not to close the straps too tightly to avoid constriction.

Precautions :

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs.

Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions :

- See product label.
- Remove metal stays before washing.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water.
- Dry away from heat sources.

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**THUASNE SAS – 118 rue Marius Aufan – CS 10032