THUASNE

Ligaflex® pro

Quick
lacing system

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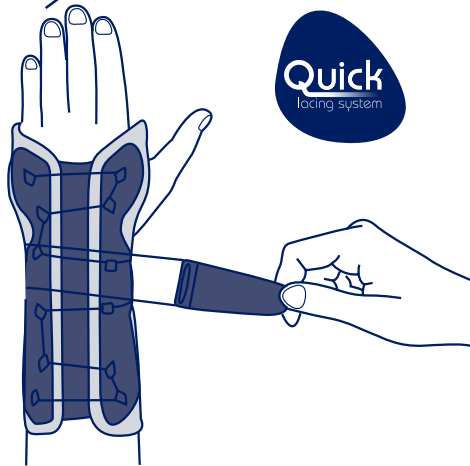
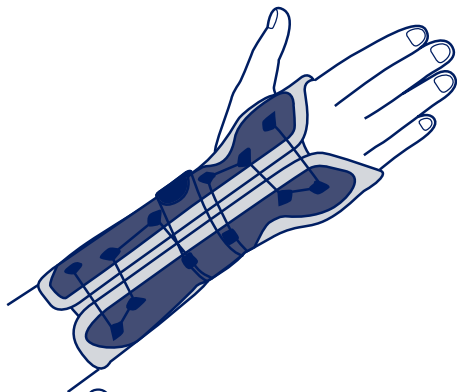
WRIST IMMOBILISATION SPLINT

Properties:

- Effective immobilisation of wrist (one anatomical and conformable X-ray transparent stay).
- "Quick Lacing System" for easy, uniform tightening.
- Aerated, foam-lined fabric ensures optimum comfort.

Indications:

- Mild sprains of the wrist (functional treatment).
- Tendinitis and tendon disorders of the wrist.
- Inflammatory phases of rheumatic diseases.
- Chronic, post-traumatic or post-operative irritations.
- Arthritis and Osteoarthritis of the wrist.

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

This is an ambidextrous wrist brace.

- Release the lace to slip the hand into the splint.
- Tighten the brace using "Quick Lacing System" until the desired level of support is obtained. If necessary, adjust the protective jersey lining when tightening.
- To avoid constriction be careful not to close the lace too tightly.

Precautions:

Follow the advice of the health professional who prescribed or supplied the product. In case of discomfort, numb fingers, impairment, loosen the brace and consult the health professional who supplied the product.

Store at room temperature, preferably in its original packaging.

For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- See product label.
- Remove the stay before washing.
- Fasten Velcro tab before washing.
- Machine wash at 30°C, delicates cycle, preferably in a wash bag.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

Composition :

See product label.

The presence of resistant laces means that the splint must not be worn close to machines with uncovered moving parts.

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Thuasne UK Ltd - Tunbridge Wells