THUASNE

MalleoDynastab®

Chevillère stabilisatrice

Stabilising ankle brace

Stabilisierende Knöchelbandage

Stabiliserende enkelbrace

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Tobillera estabilizadora

Artelheira estabilizadora

Stabiliserende ankelstøttebind

Nilkkatuki

Stabiliserande ankelortos

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STABILISING ANKLE BRACE

Properties:

- Rigid side reinforcements to ensure ankle stability. (1)
- Straps that optimise the action of the reinforcements. (2)
- Fast, easy lace fastening. (3)
- Slim product that fits easily inside a shoe.
- Anatomical shape. (4)

Indications:

- Mild, moderate or severe sprains of the lateral ligaments.
- Lower tibiofibular sprain.
- Resumption of sports.

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

- Slip the brace over the foot.
- Pull the laces to close the brace, without tightening excessively to avoid any tourniquet effect. (A)
- Cross the straps over the top of the foot once, then cross them a second time at the base of calf before fastening the Velcro ends to the sides of the ankle brace. (B-C)
- In the event of discomfort in the ankle bone area, the position of the frame can be modified, by sliding it above or below the stitch located at the bottom of the frame cover.

Precautions:

Follow the advice of the health professional who prescribed or supplied the product. Wearing this product, as with any compression product, could, in some persons, cause cutaneous reactions (redness, erythema). Should this occur, it is preferable to readjust or remove the product and ask the professional who delivered the product for advice.

Store at room temperature, preferably in its original packaging.

For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions:

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Remove the side reinforcements before washing the brace.
- Squeeze out excess water.
- Dry away from heat sources.
- N.B.: when putting the reinforcements back in place after washing, be careful to position them correctly under the grey neoprene fabric.

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Thuasne UK Ltd - Tunbridge Wells