ENGLISH

**INTENDED USE:**

Eclipse and Eclipse EXT cervical collars are intended to stabilize and restrict the motion of the cervical spine. The braces should exclusively be used to care for cervical spine pathologies.

**INDICATIONS:** Pre-operative, post-operative or post-traumatic injuries of the cervical spine, inflammatory pathologies, radicular pain, sprains, or cervical lesions.

**CONTRAINDICATIONS:** Unstable cervical fractures. Do not use on damaged or fragile skin.

**SIDE EFFECTS:** None known to this day.

**WARNINGS AND PRECAUTIONS:** If you experience pain, swelling, sensation changes, any unusual reactions, or prolonged skin redness while using this product, consult your medical professional immediately.

**NOTICE:** This device is intended to be applied by a qualified healthcare provider at the direction of a physician or licensed medical practitioner. While every effort has been made using state-of-the-art techniques to obtain maximum compatibility of function, strength,

durability and comfort, this device is only one element in the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through use of this device. Thuasne disclaims any liability for any injury or health issue by reason of application of this device.

**SINGLE PATIENT USE ONLY.**

**MATERIAL COMPONENTS:** Shell/Straps/Selector Buttons: Thermoplastic. Liner: Nylon/Polyester fabric and urethane foam. Not made with Natural Rubber Latex.

**CURRENT LEGAL REGULATIONS APPLY**

**WARRANTY:** Thuasne USA warranties all its products from the original date of purchase against defects in materials and workmanship. Normal wear and tear during use of a product is not considered a defect. Contact Thuasne USA at 855.784.6600 for specific product warranty information.

**Outside the USA: please contact the provider that delivered your brace.**

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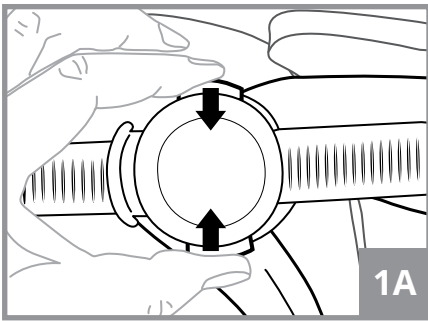
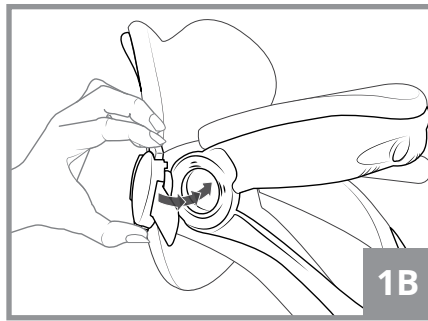
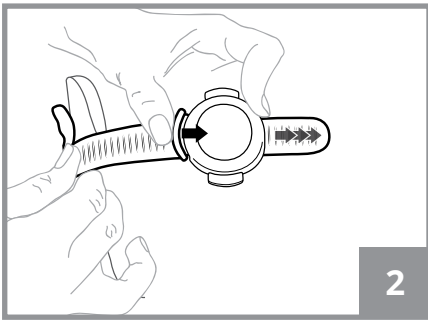
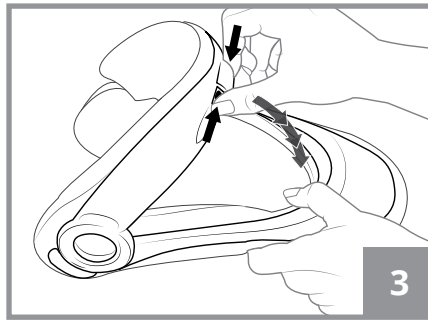
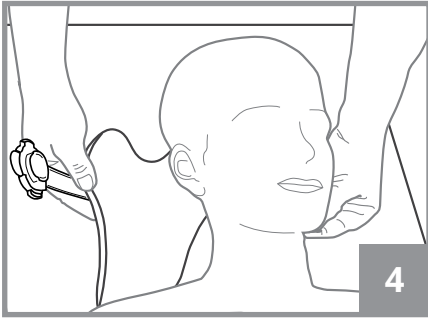
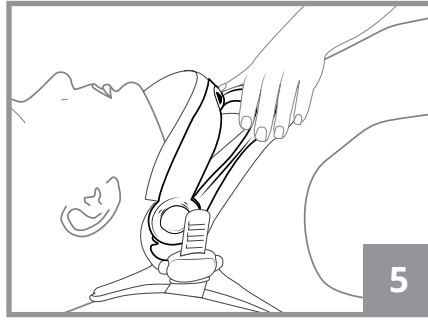
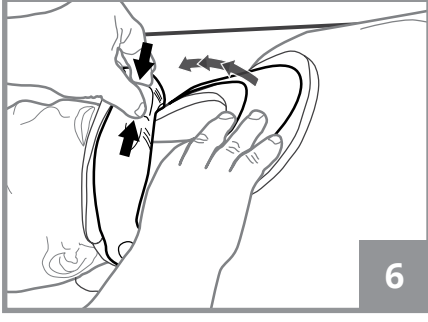
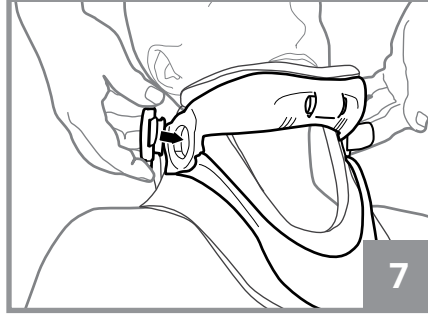
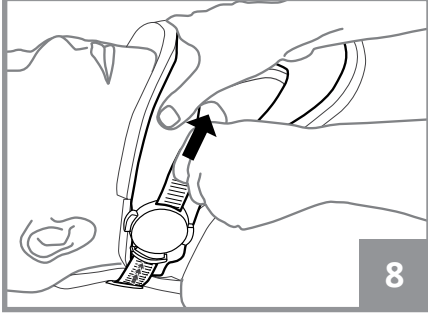
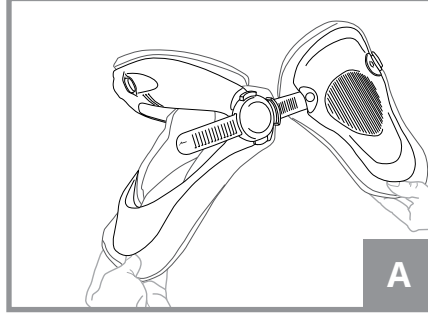
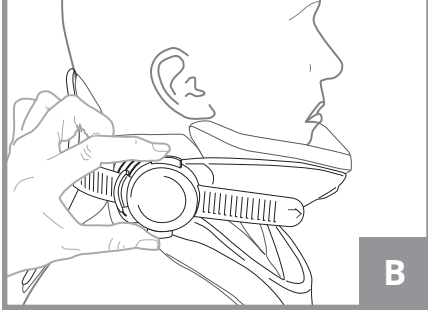
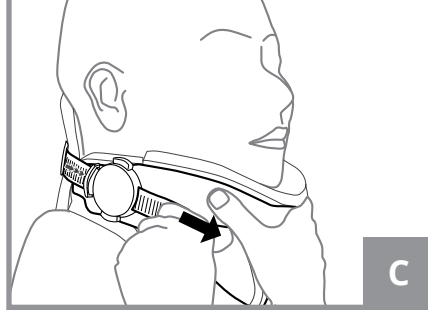
C

ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

**APPLICATION INSTRUCTIONS: SUPINE POSITION (LYING DOWN)**

- PREPARE BRACE:**
1. Remove back panel from front frame by pushing the top and bottom buttons of sizing disks (fig 1A). Lift off front frame (fig 1B). Repeat on other side.
  2. Loosen straps of back panel by pressing button located on back of sizing disk and slide sizing disk towards end of the strap (fig 2).
  3. Pinch chin buttons on front frame and lower to shortest position (fig 3).
- APPLICATION:**
1. With the patient lying down and the head in a neutral position, slide the back panel under the patient's neck (fig 4). Center the back panel with the straps between the ear and top of shoulder.
  2. Slide the front frame into position on the patient's neck (fig 5).
  3. While holding the chest piece, pinch the buttons on the chin piece and gently

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DEUTSCH

2. raise to the desired neutral position so the patient's chin rests on the foam liner (fig 6).
4. Snap the sizing disks of the back panel into the openings on both sides of the front frame (fig 7).
5. Hold the front frame by the chin piece with one hand and pull each strap forward to adjust the size until the back panel is comfortably snug against the back of the neck. The strap lengths should be even on both sides (fig 8).

**APPLICATION INSTRUCTIONS: SEATED POSITION**

- PREPARE BRACE:**
1. Remove back panel from front frame on one side by pushing the top and bottom buttons of sizing disk (fig 1A). Lift off front frame. (fig 1B, A).
  2. Loosen the straps by pressing the button located on back of sizing disk and slide sizing disk towards end of the strap (fig 2).
  3. Pinch chin buttons of front frame and lower to shortest position.
- APPLICATION:**
1. With the patient sitting and head in a neutral position, slide the front frame into position on the patient's neck. Secure the back panel to the front frame by snapping in the sizing disk on the opposite side (fig B).

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2. While holding the chest piece, pinch the buttons on the chin piece and gently raise to the desired position so the patient's chin rests on the foam liner (fig 6).
  3. Hold the front frame by the chin piece with one hand and pull each strap forward to adjust the size until the back panel is comfortable snug against the back of the neck. The strap lengths should be even on both sides (fig 8, C).
- NOTE: The patient should be in a neutral position and their chin should be well supported by the front frame. Ensure that the back edge of the front panel does not press on the throat and that the chin does not hang over the front of the frame. Readjust straps if needed.**

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**ADDITIONAL INSTRUCTIONS:**  
**REMOVE BRACE:** Squeeze top and bottom buttons of the sizing disk on **ONE SIDE ONLY** and open the collar. Gently remove from neck. Size is set for next application.  
**RE-APPLY BRACE:** With one side detached and head in a neutral position, slide the front frame into place supporting the chin and then wrap the back panel around and secure the other side. Straps do not need to be re-adjusted unless they are uncomfortable.  
**WIDE BACK PANEL:** Remove sizing disks from standard back and apply to wide back panel straps.  
**CLEANING INSTRUCTIONS:** Frame can be cleaned with a damp cloth. Liners can be hand washed in warm (86°F/30°C) water with mild soap. AIR DRY. Do not put liners in the dryer or use other heat source to dry.  
**NOTE: If not rinsed thoroughly, residual soap may cause irritation and deteriorate materials.**

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© 2017 ThuasneUSA