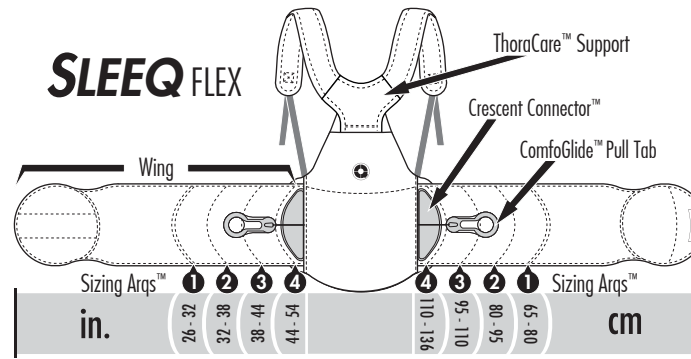
# SLEEQ

spinal • therapy • system

## INSTRUCTIONS FOR USE

### SLEEQ FLEX

**THUASNE**

4615 Shepard Street, Bakersfield, CA. 93313, USA  
800.432.3466 [ThuasneUSA.com](http://ThuasneUSA.com)

EMERGO EUROPE  
Prinsesgracht 20  
2514 AP The Hague  
The Netherlands

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DO NOT BLEACH

**Warnings**

- Carefully read use/care instructions and warnings prior to use.
- If increased pain, swelling, sensation changes, or any adverse reactions are experienced while using this product, immediately consult your medical professional.
- For single patient use only.

**Contraindications:**

- SLEEQ braces are contraindicated for women who are pregnant.
- SLEEQ braces are contraindicated for persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure would be of concern.

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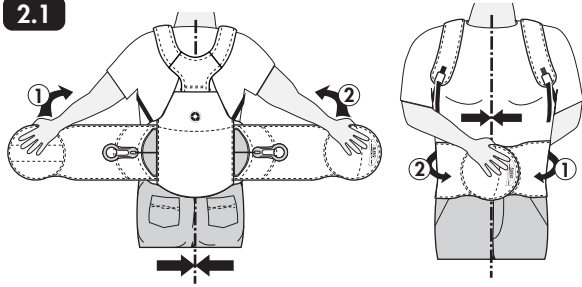
### Application

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#### 2.1

**Wrap**  
Loosen both shoulder straps and place arms through straps like putting on a backpack. ThoraCare Support may be adjusted upward prior to fitting taller patients (see 2.3 below).

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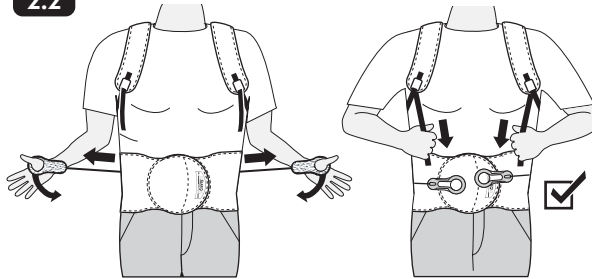
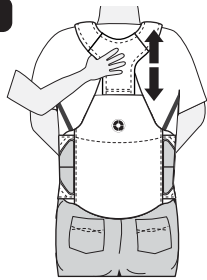
### Serrare

#### 2.3

**Adjust ThoraCare™**  
ThoraCare Support System should be adjusted upward or downward so that top center of ThoraCare support aligns at T3/scapular spine level.

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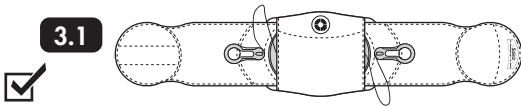
### Risistemare

#### 3.1

When removing SLEEQ, always return ComfoGlide Pull Tabs to ready position and then pull each Wing outward to reset the TruGlide™ Tightening mechanism.

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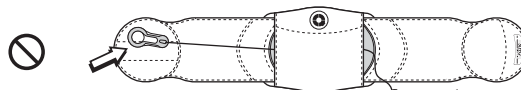
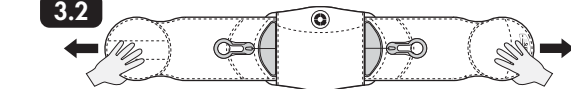
(SLEEQ #P shown)

#### LIMITED WARRANTY

Townsend Design warrants to the original purchaser that this product is free from defects in materials and workmanship. Normal wear and tear during the use of a product is not considered a defect. This Limited Warranty is effective from the date of delivery to the original purchaser.

This warranty does not apply if this product:

- (1) has been altered in any way;
- (2) was not purchased from Townsend Design or one of its authorized distributors;
- (3) is not used in accordance with Townsend Design's Instructions for Use (IFU).

Under this Limited Warranty Townsend Design's sole obligation shall be to repair or replace the defective product or associated part(s) at no charge. This Limited Warranty does not cover damage as a result of mishandling, accidents, improper use or alteration of the product. This warranty is in place of all other warranties, either express or implied, including but not limited to warranties of merchantability or fitness for a particular purpose.

Townsend Design shall not be liable under this or any implied warranty for any personal injury, property damage, or special, incidental, or consequential damages arising out of, related, or incidental to the use of the product even if Townsend Design has been informed of the prospect of such possible loss or damage, unless state law otherwise prohibits this exclusion. User should seek advice from their healthcare provider immediately if user encounters any kind of adverse reaction related to the use of this product. User should consult with their healthcare provider with any questions or concerns relating to the proper use of this product for their specific medical condition. **IMPORTANT:** Read all Instructions for Use (IFU) and any other associated materials to ensure proper use.

A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.