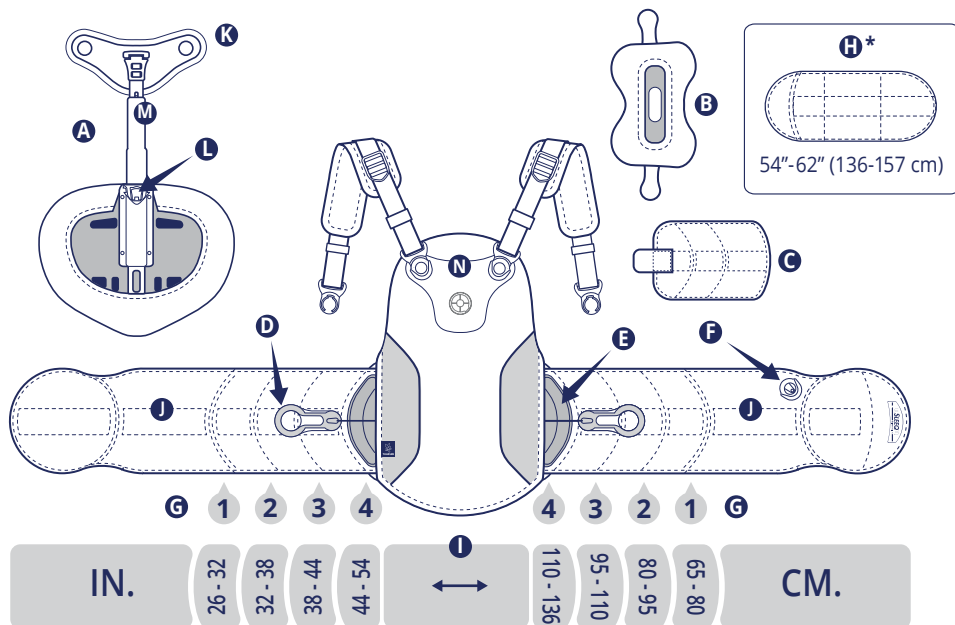
THUASNE

SLEEQ[®] MAX

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- E** VELCRO[®] CONNECTOR
- F** STRAP CLIP
- G** SIZING ARCS
- H** EXT EXTENSION PANEL
**(Sold Separately)*
- I** SIZING GUIDE

- J** WING
- K** STERNAL PLATE
- L** STERNAL BAR CLIP
- M** STERNAL BAR
- N** STRAP ANCHOR

INDICATIONS:

Sagittal/coronal/transverse plane motion restriction; stable vertebral fracture; increased intracavitary pressure to reduce spine & disc loading; mechanical or discogenic thoracic/lumbar & extremity pain; post-operative spinal stabilization/fusion; degenerative disc disease; post-injury to facilitate healing; spondylolysis; spinal stenosis; spondylolisthesis; spondylosis; laminectomy; discectomy; facet syndrome; spinal muscle weakness; arthritis; low back sprains and strains; prophylactic back support.

CONTRAINDICATIONS:

- Pregnancy.
- Persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure would be of concern.

WARNINGS:

- Carefully read use/care instructions and warnings prior to use.
- If increased pain, swelling, sensation changes, or any adverse reactions are experienced while using this product, immediately consult your medical professional.

CARE INSTRUCTIONS:

HAND WASH

HANG OR LAY FLAT TO DRY

DO NOT DRY CLEAN

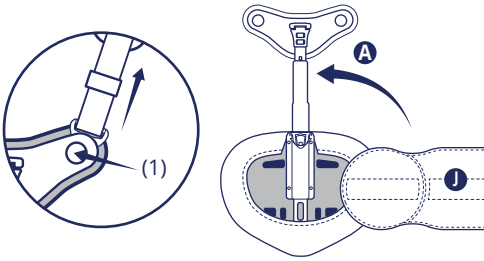
DO NOT BLEACH


DO NOT IRON

2.0 SIZING

- 2.1 PREPARE TO FIT:** Push button (1)
Remove anterior panel (A).

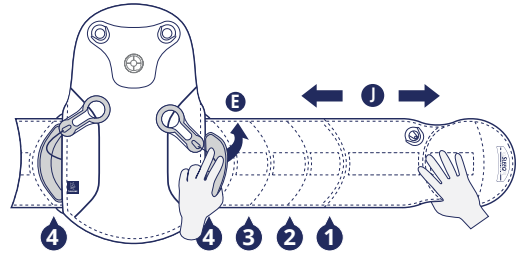
- 2.2 ADJUST CIRCUMFERENCE:** Measure waist and see Sizing Guide (1.0).

3.0 ATTACH WING SHELLS & LATERAL PANELS

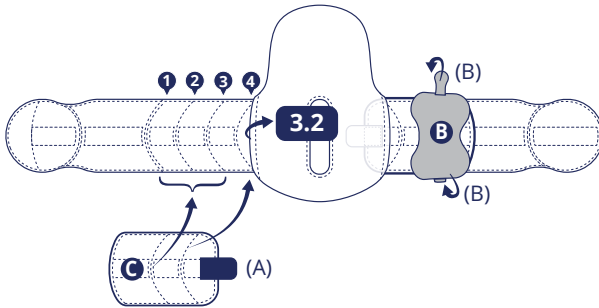- 3.1 ALIGN** wing shell (C) sew line Arcs to sew line Arcs on wing. **Note:** For size (4) use posterior Arc near Velcro tab (A).

3.2

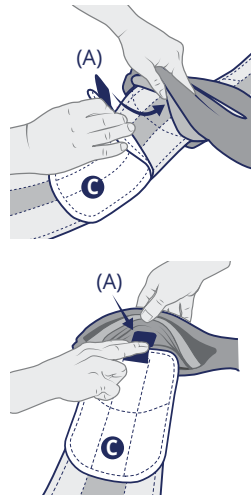
TUCK wing shell (C) inside back panel.

3.3

SECURE tab (A) to wing inside back panel. - Repeat on opposite side.

3.4


ALIGN Lateral Panel (B). Attach by folding over both tabs (B).

4.0 ADJUST AND FIT


- 4.1 POSITION** brace at coccyx level.

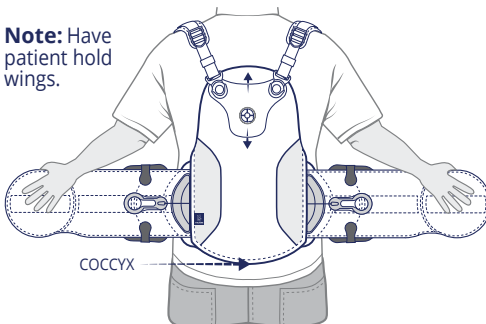Note: Have patient hold wings.

- 4.2 POSITION** Anterior Panel (A).

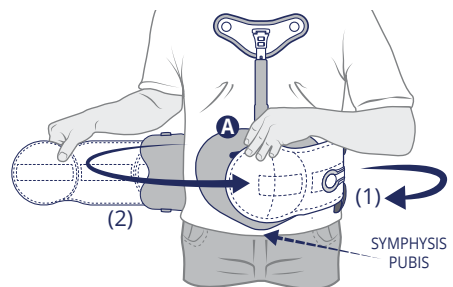Wrap wing (1) and attach to anterior panel. Wrap wing (2) and secure.

4.0 ADJUST AND FIT (Continued)

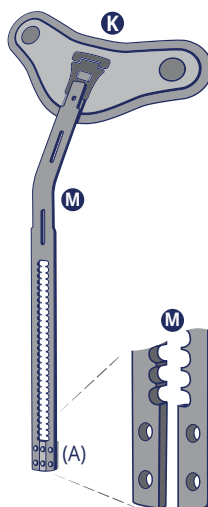
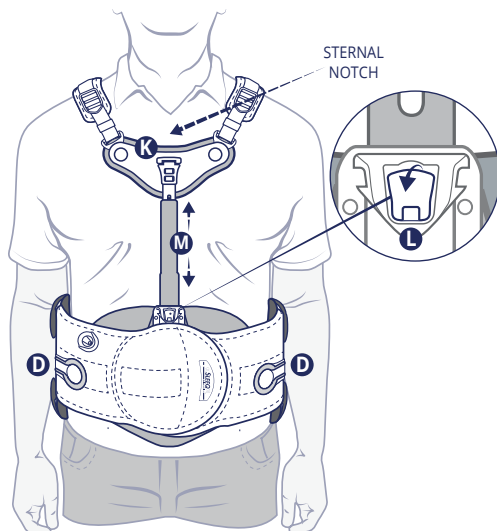
4.3 TIGHTEN BRACE: Move pull-tabs **D** anteriorly and re-attach.

4.4 STERNAL BAR: Open sternal bar clip **L** to adjust height, then close clip to lock. Sternal plate **K** is 1 to 2" (25-50cm) below Sternal Notch when standing and 1/2" (23cm) below when seated.

Sternal bar **M** can be bent to achieve proper fit with bending irons or over the edge of a table.

Note: Only bend Sternal Bar above toothed area.

4.5 REMOVING OR REPLACING STERNAL BAR (with available Short Bar option)

- Remove anterior panel from wing.
- Release sternal bar clip **L** and lower bar.
- Remove retainer clip **(A)** by pushing through bar while bending bar away from panel.
- Pull bar **M** up to remove from bar slot.
- Replace bar, or *(optional Short Bar)* into bar slot and replace retainer clip **(A)**.

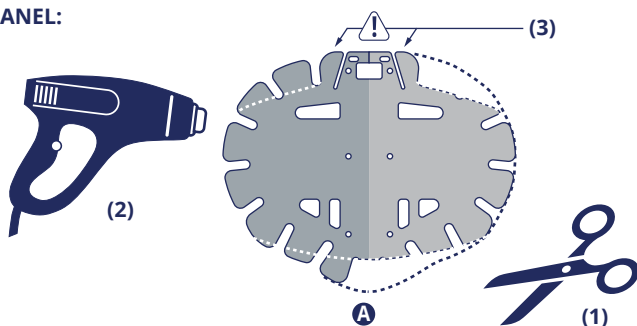
Note:
Clip **(A)** must be replaced on the panel side of bar.

5.0 Modifications

The Back Panel, Anterior Panel **A** (shown), Lateral Panels and Wing Shells can be trimmed with scissors **(1)** and heat modified **(2)** to adjust lordosis, add scapula reliefs or create flares. The steel Sternal Bar **M** can be bent to match anatomical contours and the wings can be attached to the Velcro® Connectors at up to 5 degree angles up or down to accommodate various waist to hip ratios.

CAUTION: Do not trim top center tabs **(3)** on anterior panel.

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
5.0 Modifications (Continued)


5.2 BACK PANEL REMOVAL FOR MODIFICATION:

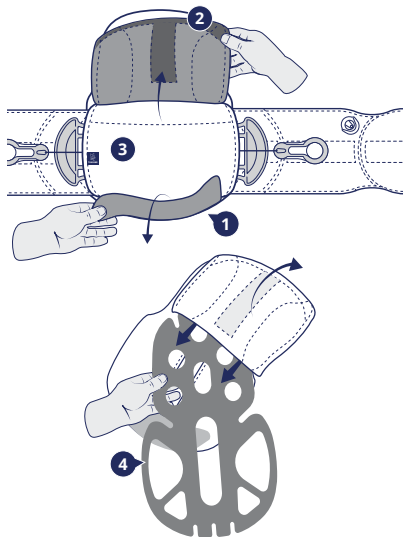- (1) Open cover at the base of the back panel. (2) Lift cover. (3) Remove the LSO assembly. (4) Remove plastic panel.

5.3 ADJUST FOR OVER THE SHOULDER OR UNDER AXILLA STRAPPING OPTION

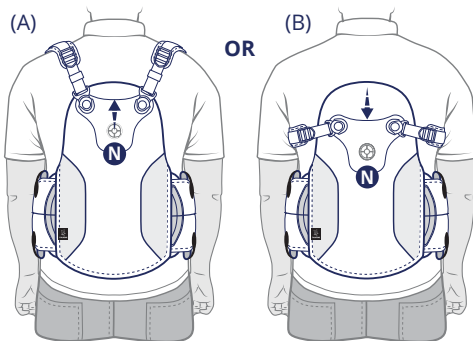
Detach Strap Anchor **N** and reposition up for over the shoulder (A) or down for under axilla strap placement (B).


6.0 Reset

When removing the brace, always replace **Pull Tabs D** to ready position (6.1). Lay brace on a flat surface then pull both **Wings J** outward to reset the tightening mechanism (6.2).

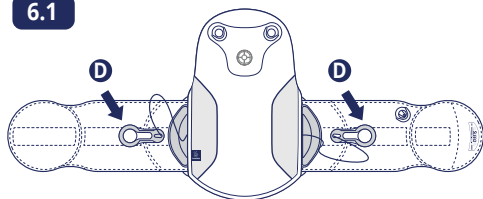6.1

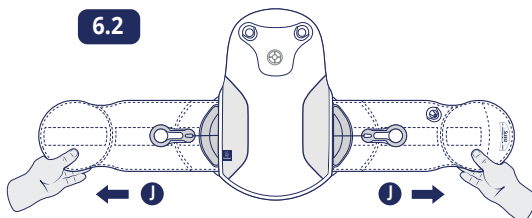6.2

Limited Warranty

Thuasne USA warrants to the original purchaser that this product is free from defects in materials and workmanship. Normal wear and tear during the use of a product is not considered a defect. This Limited Warranty is effective from the date of delivery to the original purchaser for up to 6 months.

This warranty does not apply if this product; (1) has been altered in any way; (2) was not purchased from Thuasne USA or one of its authorized distributors; (3) is not used in accordance with Thuasne USA's Instructions for Use (IFU).

Under this Limited Warranty Thuasne USA's sole obligation shall be to repair or replace the defective product or associated part(s) at no charge.

This Limited Warranty does not cover damage as a result of mishandling, accidents, improper use or alteration of the product. This warranty is in place of all other warranties, either express or implied, including but not limited to warranties of merchantability or fitness for a particular purpose. Thuasne USA shall not be liable under this or any implied warranty for any personal injury, property damage, or special, incidental, or consequential damages arising out of, related, or incidental to the use of the product even if Thuasne USA has been informed of the prospect of such possible loss or damage, unless state law otherwise prohibits this exclusion. User should seek advice from their healthcare provider immediately if user encounters any kind of adverse reaction related to the use of this product. User should consult with their healthcare provider with any questions or concerns relating to the proper use of this product for their specific medical condition. **IMPORTANT-** Read all Instructions for Use (IFU) and any other associated materials to ensure proper use.

A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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Thuasne Bégat AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.