
THUASNE

Silistab® Genu

- fr Genouillère à guide amortisseur rotulien ouvert
- en Knee brace with open shock-absorbing patellar guide
- de Patellabandage mit offenem Silikonring
- nl Knieverband met open schokdempende ring voor de knieschijf
- it Ginocchiera con guida rotulea di protezione con foro aperto
- es Rodillera con guía rotuliana protectora abierta
- pt Joelheira com guia amortecedor rotuliano aberto
- ja 膝蓋骨部ショック吸収体オープンタイプ ニープレス
- da Knæbind med åben aflastning af patella
- fi Polvituki avoimella polvilumpion suojuksella
- sv Knäortos med stötdämpande öppet patellastöd
- el Επιγονάτιδα με ανοικτό επιγονάτιο αντιδονητικό οδηγό
- cs Kolenní ortéza s otvorem pro čěšku
- pl Orteza stawu kolanowego stabilizująca z odsłoniętą rzepką
- lv Ceļa pārsējs ar atvērtu ceļa gala amortizējošu regulētāju
- it Kelio įtvaras su kelio girnelės atvira amortizacine pagalvėle
- et Avatud põlvekedratoega põlveortoos
- sl Kolenski opornica z odprtim blažilnim vodilom za pogačico
- sk Otvorená kolenná bandáž s tlmiacou vložkou okolo jabĺčka
- hu Térdrögzítő nyitott, ütécscillapító térdkalácsvédővel
- bg Колянна ортеза с отворен силиконов пръстен на капачката
- ro Orteză de genunchi mobilă
- ru Ортез для коленного сустава с открытым направителем противоударным пателлярным
- zh 减震型髌骨引导护膝 (开口型)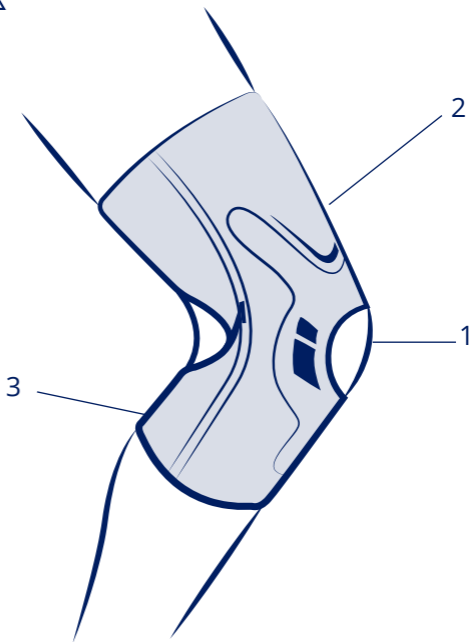

	31 - 33 cm	1
	34 - 36 cm	2
	37 - 39 cm	3
	40 - 42 cm	4
	43 - 45 cm	5
	46 - 48 cm	6

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Dispositif médical

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Conserver cette notice.

KNEE BRACE WITH OPEN SHOCK-ABSORBING PATELLAR GUIDE

Properties :

- Elastic fabric.
- Patellar opening. (1)
- Y-shaped inserts for an improved proprioceptive effect and intermittent stimulation. (2)
- Flexible lateral stays. (3)
- Comfortable lining inside.
- Anti-slip zone.

Indications :

- Patellar disorders (patellar syndromes, subdislocation and dislocation sequelae).
- Analgesic knee compression in painful or rheumatic disorders.
- Protection of injured or exposed joints.

Contraindication :

Do not apply the product in direct contact with broken skin.

Side effects :

No side effects are as yet known for correct use.

Fitting instructions :

Make sure that the kneecap is correctly centred in the patellar guide.

Precautions :

Follow the advice of the health professional who prescribed or supplied the product. Wearing this product, as with any compression product, could, in some persons, cause cutaneous reactions (redness, erythema). Should this occur, it is preferable to readjust or remove the product and ask the professional who delivered the product for advice. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient.

Washing instructions :

- See product label.
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Squeeze out excess water.
- Dry away from heat sources.

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Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne Cervitex Ltd
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.