

Indications:


Plantar Fasciitis, Achilles Tendonitis, Drop Foot, Tight Heel Cords, Dyskinesia Prevention.

Contraindications:


- Not to be used on patients with ulcerations, burns, or skin grafts of the foot.
- Not to be used on patients that have had an Achilles rupture that has been surgically repaired in the last 16 weeks.
- This device is not designed to treat uncontrollable, severe plantar flexion spasticity.

Warning:

- This is not a weight-bearing device. Do not walk or attempt to place weight on the foot while using this device.
- Should not be used to treat flexion contractures greater than 10°
- Do not pull strap too tightly
- In the event of discomfort or numbness of any kind while using this device, consult your health care provider immediately.
- The patient should be instructed to inspect their foot, ankle, and product before and after each use.
- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products

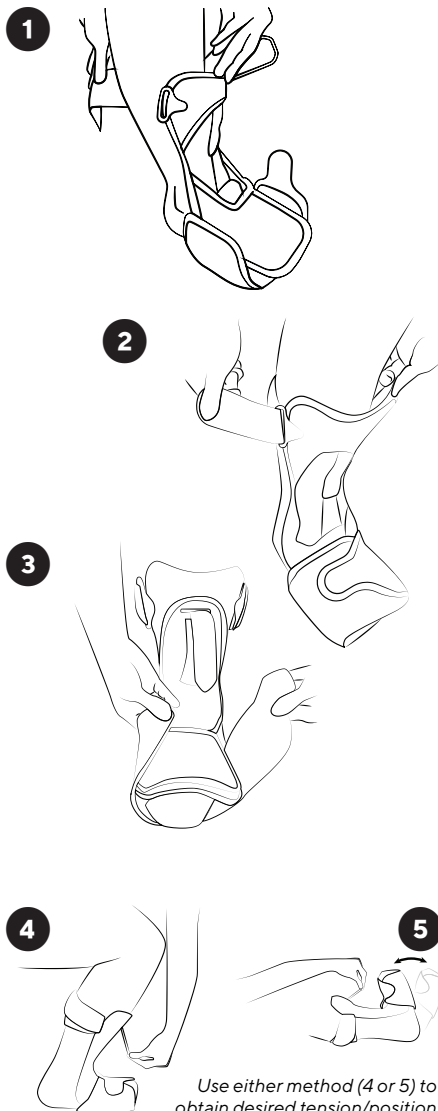


Use either method (4 or 5) to obtain desired tension/position

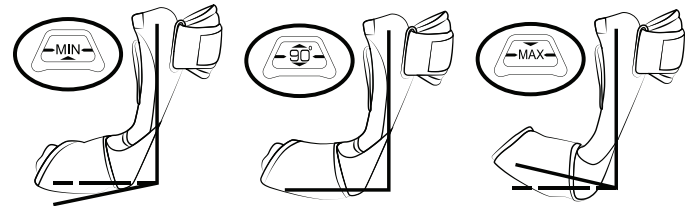

Use position indicator to measure progress

Nox Splint Accessory Strap Utilization

Uses for accessory strap are not limited to these options. Patient can customize use of strap as needed or as directed by your health care provider.

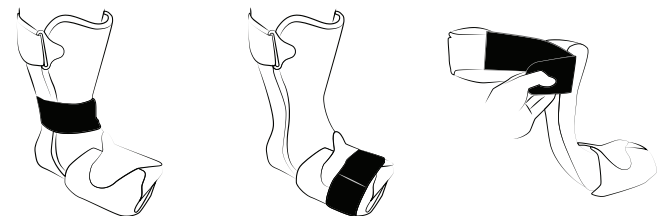

To keep splint from shifting while sleeping...

For more support around the foot...

For larger calves...

Washing Instructions:

- Do not machine wash or tumble dry.
- Do not bleach.
- Line dry only
- Avoid direct heat or sun.

Distributed by
Thuasne USA

4615 Shepard Street,
Bakersfield, CA 93313
United States
Tele: 800.432.3466
Fax: 800.798.2722
ThuasneUSA.com

L-0143 REV. A