





Stat-A-Dyne Knee

Fitting Instructions


Questions? Feel free to call us at **866-236-8889**, you can also find more information on all of our products at www.lantzmedical.com.

***It is recommended to be seated or lying down while using your Stat-A-Dyne contracture management device.**

- **Adjust the length of your Stat-A-Dyne Knee so the device is the appropriate length to fit your leg (1).**
- **While sitting, keep the leg in a neutral position and slide the brace into place. The hinge of the device should line up evenly with your knee (2). You should be sitting on top of the Upper leg cuff when the brace is fit correctly (3).**
- **Secure straps on both the Upper and Lower leg cuffs, it is not important which straps are tightened first.**

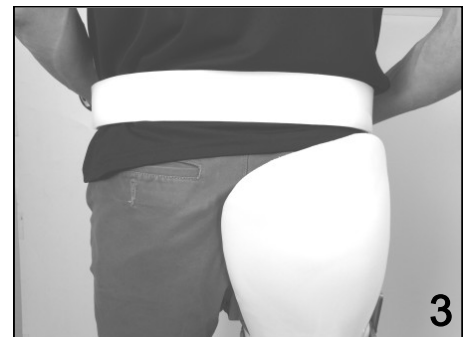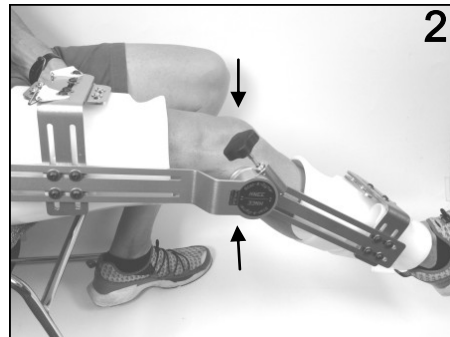

- **Secure anti-migration strap around your waist (4).**
- **You are now ready to begin your treatment session.**

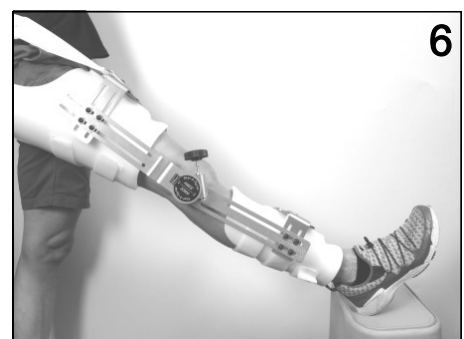

***The Stat-A-Dyne Knee is able to treat both Flexion and Extension of the Knee. Simply adjust the knob in the direction that you would like to treat. (5 flexion, 6 extension)**