



ThuasneUSA / Lantz Medical

*Phone: 866-236-8889

*Fax: 877-406-4872

* Web: www.lantzmedical.com

Stat-A-Dyne Wrist

Fitting Instructions

Questions? Feel free to call us at **866-236-8889**, you can also find more information on all of our products at www.lantzmedical.com.

***It is recommended to be seated or lying down while using your Stat-A-Dyne contracture management device.**

- There are 2 models of the Stat-A-Dyne Wrist device that are available for purchase – Left and Right. Depending on your device, follow the appropriate instructions as provided below.
- Orient the Stat-A-Dyne device by pointing the black adjustment knob towards the floor (refer to pic 1, right-hand device). Or orient the Stat-A-Dyne device by pointing the black adjustment knob towards the sky/upward direction if your device is a left-hand Stat-A-Dyne device.
- Align the Stat-A-Dyne hinge through the center of your wrist by sliding your arm through the forearm cuff (refer to pic 5).
- If necessary, adjust the length of the finger platform by loosening the 3 screws and elongating the device (refer to pic 2).
- The forearm splint can be adjusted by loosening the 2 wing nuts located on the lateral side of the brace. With the arm bent at 90°, there should be 2 finger widths between the proximal end of the forearm cuff and the elbow crease (refer to pic 3).
- Once the device is the appropriate length, loosen the straps, slide your forearm through the cuff and hand strap (refer to pic 4).
- Position your forearm in the device so your wrist is lined up with the center of the hinge on the device (refer to pic 5).
- Tighten the forearm and palm straps (refer to pic 6).
- You are now ready to begin your treatment session.

***The Stat-A-Dyne Wrist device is able to treat both extension and flexion when fit following the instructions on this guide.**

