







# Casting Platform

## Product Details

assists in capturing a plaster negative of the foot with various heel heights and adjustable toe pitch in the forefoot area, appropriate for AFO, KAFO.

## Scope of Delivery

- 1x Foot/ Casting Base Plate with Guide Rods
- 1x Adjustable Toe Ramp/ Forefoot wedge (10 different Positions)
- Heel Adjustment Blocks - (2) 1", (1) ½", (1) ¼" (2) ⅞"

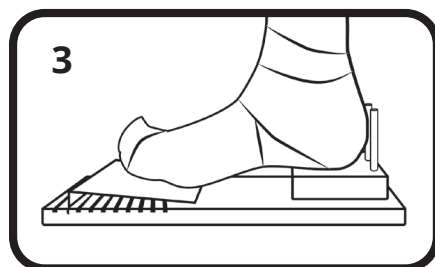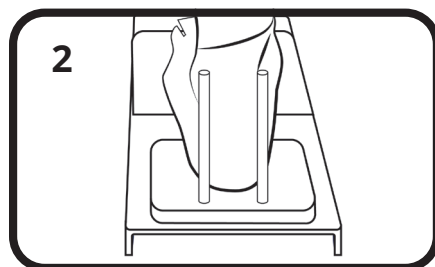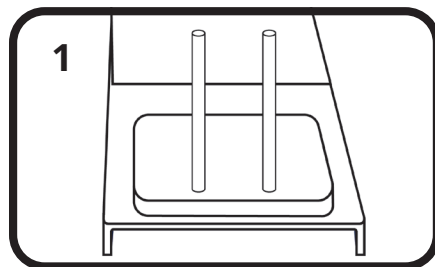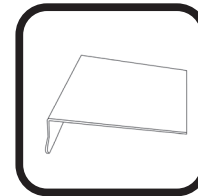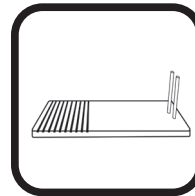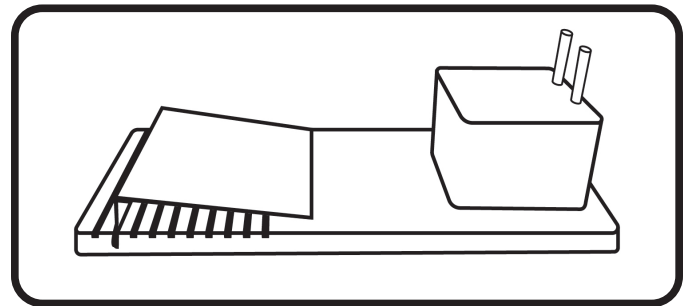

## Directions for Use

1. Using anchor posts on footplate, position Heel block to match the profile of the intended footwear
2. Make sure Hindfoot is neutral and square to anchor posts on footplate. The anchor posts are just reference points, do not weight bear or let cast rest on these, as they could cause a cast deformity.
3. Position 1st & 5th head of metatarsals level in horizontal plane with front edge of toe ramp (adjustments may need to be made to accommodate foot size)

**\* To avoid cast deformity do not position heel/ ankle in direct contact with posts**

## Information & Tips

- Maximum Heel Height is 3 inches
- Determine the toe pitch and heel height before taking plaster cast

## Cleaning

If necessary, all components can be cleaned with soap and water and then wiped dry. Lubrication of sliding components may be needed from time to time.

## Warranty

Manufacturer's warranty applies only if this device has been used under the conditions for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

**Distributed by  
Thuasne USA**

4615 Shepard Street,  
Bakersfield, CA 93313  
United States  
Tele: 800.432.3466  
Fax: 800.798.2722  
[ThuasneUSA.com](http://ThuasneUSA.com)

L-0100 REV. B



Townsend Design  
4615 Shepard Street  
Bakersfield, CA 93313  
United States  
Tel 661.837.1795