THUASNE

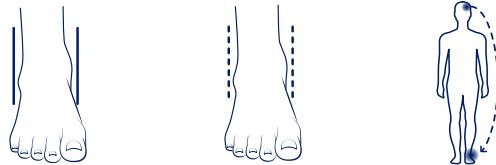
MalleoDynastab® BOA®

fr Chevillère d'immobilisation stabilisatrice avec système de serrage BOA Fit System	6
en Stabilising ankle-immobilising brace with BOA Fit System closure	7
de Stabilisierende Sprunggelenkorthese zur Ruhigstellung mit BOA Fit System zum Festziehen.....	8
nl Stabiliserende immobilisatiebrace voor de enkel met BOA Fit System-klemsysteem	10
it Cavigliera d'immobilizzazione stabilizzatrice con sistema di chiusura BOA Fit System	11
es Tobillera de inmovilización estabilizadora con sistema de ajuste BOA Fit System	13
pt Artelheira estabilizadora de imobilização com sistema de aperto BOA Fit System	14
da Immobiliserende ankelstøttebind med BOA Fit System til stramning	15
fi Immobilisoiva vakauttava nilkkatuki, jossa on BOA Fit System -kiristysjärjestelmä	17
sv Stabiliserande och immobiliserande ankelortos med fästsystemet BOA Fit System	18
el Σταθεροποιητική επιστραγαλίδα ακινητοποίησης με BOA Fit System	20
cs Stabilizační ortéza na kotník se stahovacím systémem BOA Fit System	21
pl Stabilizująca orteza skokowo-stopowa z systemem zaciskania BOA Fit System	22
lv Stabilizējoša potītes imobilizācijas ortoze ar BOA Fit System	24
lt Stabilizuojantis imobilizuojantis čiurnos įtvaras su „BOA Fit System” suveržimo sistema	25
et BOA Fit System kinnitussüsteemiga stabiliseeriv fikseeriv hüppeliigese ortoos	26
sl Opornica s stabilizatorjem za gleženj s sistemom vpenjanja BOA Fit System	28
sk Imobilizačná členková ortéza s upínacím BOA Fit Systémom	29
hu Stabilizáló bokarögzítő BOA Fit System szorítórendszerrel	31
bg Обездвижващ стабилизатор за глезен със система за затягане BOA Fit System	32
ro Gleznieră stabilizatoare de imobilizare cu sistem de strângere BOA Fit System	33
ru Ортез стабилизирующий голеностопный с системой фиксации BOA Fit System	35
zh 带BOA Fit System闭锁系统的稳定式护踝	36
ar BOA Fit System رباط لشیت وحماية الكاحل ومنزود بنظام ربط	39

	O	16 - 19 cm
	1	19 - 22 cm
	2	22 - 26 cm
	3	26 - 30 cm

fr	Tour de cheville
en	Ankle circumference
de	Knöchelumfang
nl	Orvango van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Niilkanypärys
sv	Runt fotleden
el	Περιμέτρος αστραγάλου
cs	Hleženní stabilizační ortéza

pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
si	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferință gleznei
ru	Окружность голени
zh	脚踝尺寸
ar	الارتفاع على خط


fr	Immobilisation	Stabilisation	Proprioception
en	Immobilization	Stabilization	Proprioception
de	Ruhigstellung	Stabilisierung	Proprioception
nl	Immobilisatie	Stabilisatie	Proprioception
it	Immobilizzazione	Stabilizzazione	Propriocezione
es	Inmovilización	Estabilización	Propriocepción
pt	Imobilização	Estabilização	Propriocção
da	Støtte	Stabilisering	Proprioception
fi	Immobilisoi	Stabiloii	Proprioceptio
sv	Immobilisering	Stabilisering	Proprioception
el	Ακινητοποίηση	Σταθεροποίηση	Ιδιοδεκτικότητα
cs	Znehýbnění	Stabilizace	Propriocepce
pl	Unieruchomienie	Stabilizacja	Propriocepja
lv	Imobilizācija	Stabilizēšana	Propriocepja
lt	Imobilizavimas	Stabilizavimas	Propriocepja
et	Liikumatuks muutmine	Stabiliseerimine	Proprioceptsioon
sl	Imobilizacija	Stabiliziranje	Proprioceptivni dražljaji
sk	Imobilizácia	Stabilizácia	Propriocepacia
hu	Rögzítés	Stabilizálás	Propriocepció (helyzetérzékelés)
bg	Обезгвъждане	Смабулизация	Проприоцепция
ro	Imobilizare	Stabilizare	Propriocepție
ru	Иммобилизация	Стабилизация	Проприоцепция
zh	固定	稳定	本体感受
ar	التحفيز	الاستقرار	الاعمق الحس استقبال

fr	Appuyer pour enclencher	Tournez pour serrer	Tirez pour desserrer rapidement
en	Push in to engage	Turn to tighten	Pull up for quick release
de	Zum Festziehen eindrücken	Zum Festziehen drehen	Zum schnellen Lockern herausziehen
nl	Indrukken om aan te draaien	Draaien om aan te spannen	Optrekken om snel los te maken
it	Premere per inserire	Girare per stringere	Tirare per allentare
es	Empuje para armar	Gire para apretar	Levante para soltar rápidamente
pt	Premir para acionar	Rodar para apertar	Puxar para cima para soltar rapidamente
da	Skub ind for at fastgøre	Drej for at stramme	Træk op for en hurtig løsning
fi	Kytke painamalla	Kiristä kiertämällä	Vapauta nopeasti vetämällä
sv	Tryck in för att sätta fast	Vrid för att dra åt	Dra uppåt till för att lossa snabbt
el	Πιέστε προς τα μέσα για ασφάλιση	Στρέψτε για σύσφιξη	Τραβήγτε προς τα έω για γρήγορη απελευθέρωση
cs	Stisknutím aktivujte	Otočením utáhněte	Zvednutím povolte
pl	Wcisnąć, aby wprowadzić	Przekręcić, aby zamocować	Pociągnąć w góre, aby szybko zwolnić
lv	Nospiediet, lai ieslēgtu	Pagrieziet, lai savilktu	Pavelciet, lai ātri atbrīvotu
lt	Paspauskite, kad sujungtumėtė	Pasukite, kad suveržtumėtė	Patraukite, kad greitai atlaisvintumėtė
et	Sulgemiseks vajutage	Pingutamiseks keerake	Kiireks lahti tegemiseks tömmake
sl	Pritisni, da se vklopi	Zavrti, da se vpne	Povleci gor, da se hitro odklopi
sk	Zatlačením zaistite	Otočením utiahnite	Potiahnutím rýchlo uvolnite
hu	Rögzítéshez nyomja meg	Szorításhoz fordítsa el	Gyors kioldáshoz húzza ki
bg	Натиснете, за да поставите	Завъртете, за да затегнете	Издърпайте нагоре за бързо освобождаване
ro	Apăsați pentru a cupla	Rotiți pentru a strângă	Trageți în afară pentru a elibera rapid
ru	Нажмите, чтобы перевести систему фиксации в рабочее состояние	Поверните, чтобы затянуть шнурки	Потяните, чтобы рассстегнуть
zh	向内推动实现啮合	转动收紧	向上拉起实现快速释放
ar	اضغطوا على الصمام	أدبروها لشد الشراك	اسحبوها لارتفاع سريع

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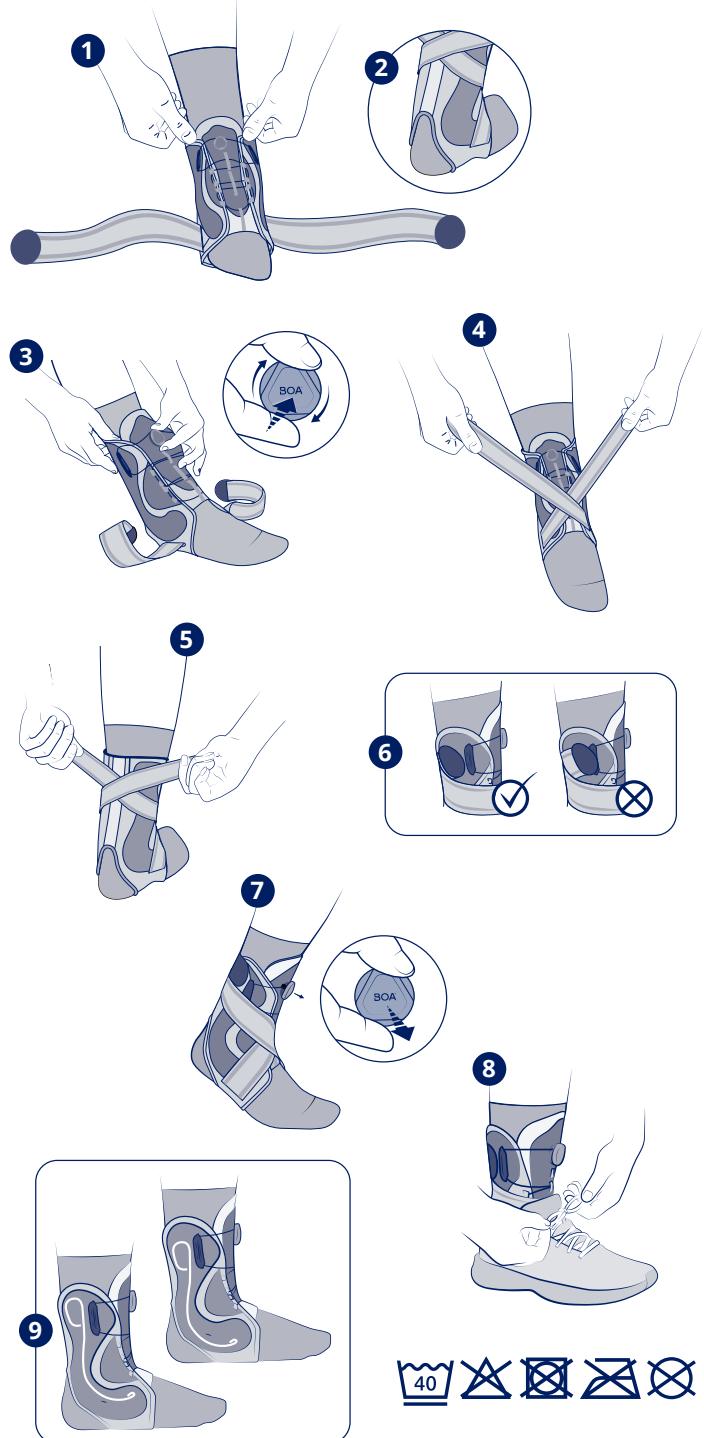Conserver cette notice.

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en

STABILISING ANKLE-IMMOBILISING BRACE WITH BOA FIT SYSTEM CLOSURE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Textile parts: neoprene, polyamide, elastane, polyester, polyethylene, polyurethane.

Elastic straps: polyester, elastane.

Self-fastening tabs: polyamide.

Plastic parts: polyamide 66.

Side reinforcements: stainless steel.

Properties/Mode of action

Stabilising ankle brace to provide immobilisation, incorporating rigid frame and elasticated straps to give stability and immobilisation to the ankle.

Fast, durable and micro-adjustable BOA Fit System for a custom fit, with a lifetime guarantee.

A slim product that will fit easily inside a shoe.

This is a bilateral product.

Indications

Ankle immobilisation.

Moderate to severe ankle sprain.

Joint laxity/chronic instability prevention during sport activities resumption.

Sport activities resumption.

Tendinopathy treatment.

Conservative treatment of Achilles tendon injuries.

Contraindications

Do not use this product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

For hygiene and performance reasons, do not re-use the product for another patient.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Do not wear the product in a medical imaging device.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

If the product comes into contact with water, make sure to remove the lateral straps. Dry out the fabric part and wipe the stays thoroughly using a dry cloth.

Undesirable side-effects

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of vein thrombosis.

Any serious incident arising that is linked to the product should form the subject of a report to the manufacturers and the competent authority for the Member State in which the user and/or patient is domiciled.

Instructions for use/Application

We recommend that the product should be worn inside a shoe by daytime whilst walking, and by night for protection in order to prevent and limit sudden movements.

Before putting the ankle brace on, make sure the laces are completely unfastened.

A. Fitting the ankle brace:

Slide the foot inside the ankle brace. Ensure that the heel is resting inside the opening provided at the rear of the product for this purpose. ① ②

B. Fastening the ankle brace:

Tighten the ankle brace using the BOA Fit System closure system; to do this, push the wheel in until you hear a 'click', then turn clockwise until the correct level of support is obtained. ③

C. Fitting the straps:

Take a strap in each hand, then pass them over each other across the upper foot. ④
(Fig. 4) Wrap around the ankle, cross the two straps over again around the rear of the heel, and attach the self-fastening tabs to the upper portion of the ankle-brace fabric. ⑤
Verify the secure grip of the self-fastening tabs over the fabric area by pressing on the self-fastening part. Do not place the self-fastening tabs over the black plastic portion. ⑥

D. Releasing the ankle brace:

Pull on the wheel until you hear the 'click' to unfasten the laces. ⑦

E. Fitting inside a shoe:

Once the ankle brace is in position, slide the foot carefully inside a closed shoe and fasten the shoe's closure mechanism. ⑧

In the event of any discomfort at the ankle-bone area, it is possible to change the position of the stays: Detach the self-fastening tabs from the black fabric area, take hold of the rigid metal part and remove it. Slide the stays above or below the stitching located either side of the ankle brace. ⑨

Care/Maintenance

Product can be washed in accordance with the conditions shown on this notice and on the label. Make sure to remove the lateral stays before washing. Replace in the original location before next use.

Close the self-fastening tabs and close the product (BOA Fit System) before washing. If possible, use a washing net.

Dry in a ventilated area, away from sources of heat.

Storage

Store at room temperature, preferably in its original packaging.

Disposal

Dispose of in accordance with local regulations.

Warranty

BOA Technology Inc. guarantees the closure system (wheel and lace) for the product's lifetime. This warranty applies for normal conditions of use of the product.

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©Thuasne - 2019302 (2022-05)