THUASNE

SLEEQ® FLEX

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Sleeq Flex (901400)

Spinal orthosis for active relief and correction of the lumbar and thoracic spine in the sagittal plane.

EN

Purpose:

The Sleeq Flex spinal orthosis is an orthosis for active relief and correction of the lumbar and the thoracic spine in the sagittal plane. The orthosis may only be used for lower spine treatment. This device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Properties

- Pulley system for individual power transmission
- Pliable material
- Reinforcing elements in the abdominal area

Indications:

- Stable, osteoporotic vertebral fractures
- Kyphosis with poor shoulder posture
- Spondylitis
- Morbus Bechterew (Arthritis)

Contraindications:

- Unstable vertebral fractures
- Do not use the product if the diagnosis has not been confirmed
- Do not use in the event of known allergy to any of the components
- Do not apply the product in direct contact with broken skin
- Pregnancy
- Persons with circulatory, cardiovascular or other conditions where increased abdominal pressure would be of concern

Side Effects:

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Putting on the Orthosis:

Size Determination and Adjustment

The SLEEQ Flex spinal orthosis is universally sized and can be individually adjusted to the patient circumferences.

(Fig.1) Four sizing arqs (1, 2, 3, 4) show markings for the waist size and the correct alignment. They are used for orientation since the size can be varied and individually adjusted.

(Fig.2) Loosen the pull tabs and place them on the spinal pad, with the hook-and-loop facing up. Adjust the right wing to the inside or the outside and firmly press the crescent-shaped hook-and-loop down on the suitable size marking (or between the markings). Repeat this setting with the left wing as well. Make sure that the hook-and-loop fastening fits firmly between the waist strap and the crescent-shaped hook-and-loop part. If the desired setting does not fit, repeat it until the circumference is correct.

(Fig.3) If necessary, the waist strap may be fixed slightly diagonally upwards or downwards to thus obtain a better waist adjustment.

In order to reset the pulley system, leave the pull tabs on the spinal pad and simultaneously pull on the left and right waist strap. Once you feel resistance, the system will be adjusted and both waist straps should now have the same length.

Putting on the orthosis

(Fig.4) Loosen the shoulder straps and put the arms through the straps (for larger patients, the "ThoraCare" carrier may be pushed upwards before putting it on). Now center the spinal pad at the desired height in the lumbar region (T-3 / scapular spine level) and then close the waist belt firmly (to do this, place the left strap on the abdomen and use the hook-and-loop system to fasten the right strap onto the left).

(Fig.5a) For the individual pressure setting, please loosen the pull tabs. Then put your thumbs into the opening and pull the straps forward to the outside until you feel a comfortable and firming pressure. In order to fix the two strap ends, simply use the hook-and-loop system to fasten the flaps again on the waist strap. In order to firm up the shoulder straps, simply pull the traction ropes downward (like a backpack system) until they rest comfortably against the shoulders.

(Fig.5b) In order to adjust the rear lengths, push the "ThoraCare" carrier upward or downward in the pocket so that the upper center of the carrier is on a level with T3.

Warnings:

Carefully read use/care instructions and warnings prior to use. If increased pain, swelling, sensation changes, or any adverse reactions are experienced while using this product, immediately consult your medical professional.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care:

Do not use any cleaning agents, conditioners or aggressive products (containing chlorine). Squeeze out the water thoroughly. Dry away from any heat sources

Material Composition:

Nylon, Polyester, Polypropylene, synthetic rubber (latex-free)

Disposal:

Dispose of in accordance with local regulations.

Keep this instruction leaflet.

Limited Warranty:

Thuasne USA will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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