

IMPORTANT: The Vector 2 device must be set to 150 degrees of flexion, before the device is prepared and boxed for return shipment.

Components



1. (1) Vector 2 Device with attached LCD hand controller.
2. (1) Palmer Plate.
3. (2) Wall power adapters.
4. (1) Battery pack nylon belt.
5. (1) Vector 2 battery pack.

Soft Good Removal



Remove and discard the (2) forearm straps.



Remove and discard the forearm pad.



Remove and discard the (4) palmer plate straps.



Remove and discard the palmer plate pad.



Remove and discard the Vector 2 glove by detaching glove away from the finger shoes.



Remove and discard the dorsal MP pad.


Lower Tray



Insert the Vector 2 battery pack into the Lower Tray.
Note: The notches in the tray for the battery pack's input / output jacks.



Insert the (2) wall power adapters in the top right square cutout section. Note: The wiring should be tucked inside the middle of the Lower Tray.

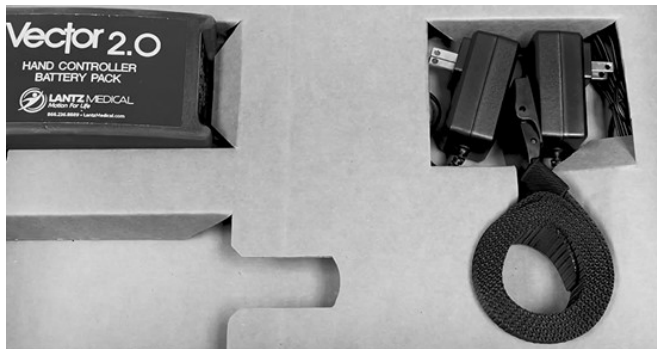

Tightly coil the battery pack nylon belt and place inside the circular cutout of the Lower Tray.



Insert the LCD hand controller, as shown. Tuck the open-ended cable inside the middle of the Lower Tray. If possible, use the original LCD bubble wrap.




Insert the palmer plate in any of the Lower Tray cutouts, except the LCD hand controller section.



The Lower Tray is now complete. Proceed to the Upper Tray.

Upper Tray

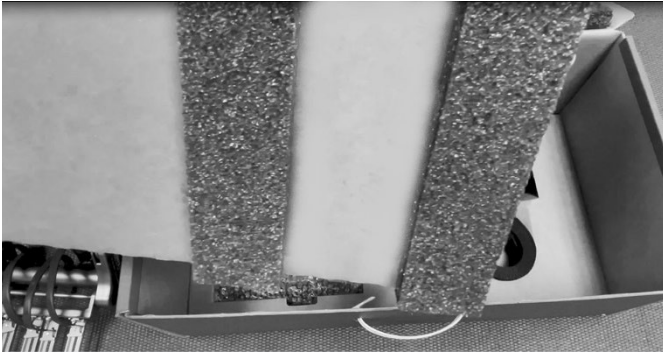

Note: The bottom side of the Upper Tray has (2) strips of foam padding.



The (2) strips of foam padding should be rest over the nylon belt and (2) wall power adapters.



Note: The corner of the Upper Tray will have a small cutout. This is where to route the LCD hand controller cable from the Lower Tray to Upper Tray.



Insert the Upper Tray, as shown. Note: The cable routing from Lower Tray. The Vector 2 Box handle will rest inside the middle of the Lower Tray.



Gently place the Vector 2 device inside the Upper Tray cutout, as shown.




Ensure the cable is not pinched by any device hardware.

Preparing Vector 2 for Return Shipment



Fold the Vector 2 Box lid over towards the front of the box, making sure the small lid flaps are tuck inside the box.

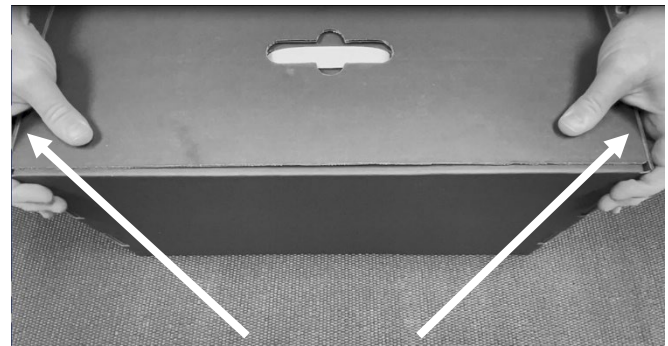

Tuck the (2) front lid flaps into the sides of the Vector 2 Box.



The Vector 2 device is almost ready for return shipment. We recommend securing the Vector 2 Box lid with (2) small pieces of shipping tape.



Tape increases protection of the Vector 2 during shipment. The Vector 2 device is now ready for return shipment to Thuasne USA / Lantz Medical.



See page 6 for proper Return Label Placement.

Vector 2 Return Label Placement



If a UPS Return Shipment Label has not already affixed for you, place the provided UPS Return Shipment Label over the original UPS Shipping Label. **Note: Return Shipment Labels are distinguished by the “RS” at the top right corner and will have a “SHIP TO:” address of SHIPPING DEPT at Lantz Medical.**

To notify us of a device RETURN, please call 317.536.4870 ext. 230 or email sprice@thuasneusa.com. Please provide the RETURN TRACKING NUMBER, patient name, and device serial number.