



ThuasneUSA / Lantz Medical

*Phone: 866-236-8889

*Fax: 877-406-4872

* Web: www.lantzmedical.com

PIP Flexion Splint Fitting Instructions

Questions? Feel free to call us at **866-236-8889**, you can also find more information on all of our products at www.lantzmedical.com.

***It is recommended to be seated while utilizing the device.**

APPLYING THE LANTZ MEDICAL PIP FLEXION SPLINT:

- Loosen the counterforce strap or open completely. Slide finger into splint so that the cams align with the PIP joint of the finger. **See Figure 1.**
- The finger cradle can be adjusted for proper fit by using a Philips head screw driver to adjust width. Place screw driver inside opening of cradle that houses the screw and turn either clockwise or counter clockwise to adjust a comfortable width. Clockwise to reduce width and counterclockwise to increase width. Secure counterforce strap under finger.
- To adjust dynamic tension turn the loading screws either clockwise to increase tension or counterclockwise to reduce tension.

REMOVING THE LANTZ MEDICAL PIP EXTENSION SPLINT:

- Loosen the counterforce strap.
- Extend unit to release tension then remove finger from splint.

Figure 1.

