





Measuring and Sizing Guide

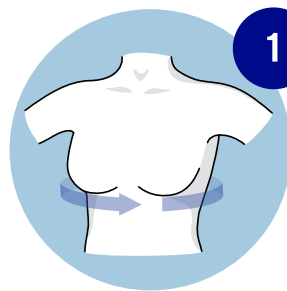

1

Band Size

Measure the circumference just under the breast for band size

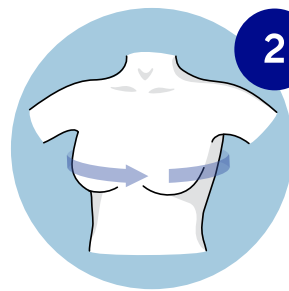

2

Cup Size

Measure the circumference of the fullest part of the breast for cup size

Sizing Chart		XS	S	M	L	XL
① Band Size	CM	68 - 77 cm	78 - 87 cm	88 - 97 cm	98 - 107 cm	108 - 117 cm
	AA/A	80 - 89 cm	90 - 99 cm	100 - 109 cm	112 - 119 cm	120 - 129 cm
② Cup Size	ITEM #	T571301201AAA24	T571301202AAA24	T571301203AAA24	T571301204AAA24	T571301205AAA24
	B/C	84 - 93 cm	94 - 103 cm	104 - 113 cm	116 - 123 cm	124 - 133 cm
	ITEM #	T5713012010BC24	T5713012020BC24	T5713012030BC24	T5713012040BC24	T5713012050BC24
	D/E	88 - 97 cm	98 - 107 cm	108 - 117 cm	120 - 127 cm	128 - 137 cm
	ITEM #	T5713012010DE24	T5713012020DE24	T5713012030DE24	T5713012040DE24	T5713012050DE24



Care Instructions

It is recommended to wash your MOBIDERM® garment daily (or after each use).

MOBIDERM® INTIMATE BRA AND PADS:

- Gentle machine wash on cold setting.
- Use mild detergents. Avoid use of bleach, fabric softener or other harsh chemicals.
- Place the garment in a wash bag with similar colors.
- Hang or lay flat to dry the bra, and keep it away from direct sunlight.
- Ensure all clasps and fasteners are secure prior to washing.
- Do not iron.



Materials

PRODUCT CONTAINS:

- 64% Nylon
- 24% Elastane
- 7% Polyester
- 5% Polyurethane

Soft & Breathable Experience!



Indications

- Acute or chronic edema and/or lymphedema of the breast and/or breast wall
- Postoperative edema or edema after radiotherapy (only on intact skin)
- Chronic infiltration after reconstruction, remaining chronic breast wall edema



Scan the QR code
to learn more about the
Mobiderm® Intimate Bra

1-800-821-3094
cs@knitrite.com
ThuasneUSA.com



MOBIDERM® Intimate Bra

Coverage You Need,
Discreetness You Desire.



MOBIDERM® Intimate Bra

The Mobiderm® Intimate Bra provides optimum care for breast related edema or lymphedema by providing coverage on the breast, thoracic wall and back.

The Mobiderm® Intimate Bra provides strong compression and integrates removable and discreet foam compression pads to create high and low pressure differentials on the affected regions, in result, softening tissue and improving lymph drainage. Moreover, this controlled compression solution helps minimize scar tissue and the development of hematomas.



Manufacturing Type: Off-The-Shelf



Compression Level: 5-10mmHg




Color: Nude



Warranty: 6 Month Manufacturer's Warranty

The MOBIDERM® Technology

Mobiderm® foam blocks provide a comfortable, massaging effect to support clinical efficacy and compliance. This technology is made up of big or small foam blocks encased between two non-woven bandages. Due to the difference in pressure created between the support area of the blocks and their surrounding area, it promotes the flow of lymphatic fluid and optimizes the drainage efficacy. The unique spacing of our Mobiderm® foam blocks also creates passage ways for air to flow to support breathability.

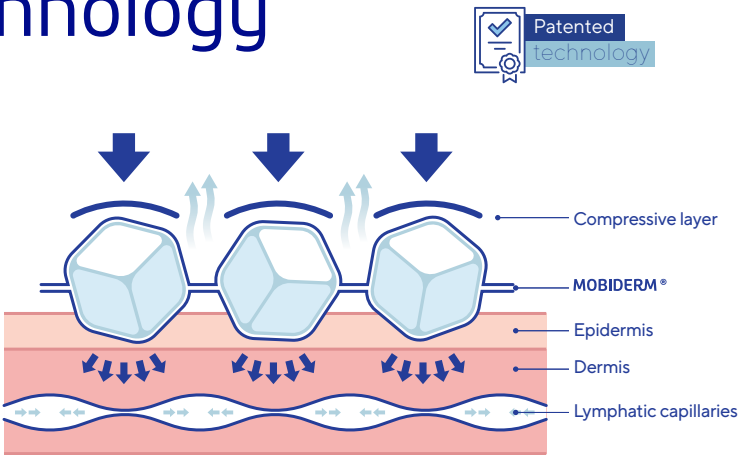

SOFT INTERIOR SEAMS

Prevents skin irritations for patient compliance

HIGH LATERAL DESIGN

Provides good coverage and added support for affected areas

WIDE BAND

Prevents the bra from flipping up during wear



ADJUSTABLE SHOULDER STRAPS

Find the best personal fit with wide padded straps for your comfort

FRONT ZIPPER WITH END COVERS

Easy to put on and take off with covers to protect skin on each end

INTEGRATED POCKETS

For inserting the breast form of your choice and included Small Block Breast Pads

COLORED HOOKS

Come in a darker color than the bra to make it easier to see where the bra clasps together



Interior Side Pockets



Interior Back Pockets



Zipper Liner

ACCESSORY



- ✓ 2 Small Block Breast Pads included
- ✓ Fits into the integrated pockets
- ✓ Provides discreet low profile therapy
- ✓ Additional Mobiderm® Pads (available in big and small block) sold separately to cut and use for different desired shapes or sizes.

